Cocoon[®] / Cocoon[®] Spray

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C O N T E N T S

I - PRESENTATION AND INSTALLATION

Presentation	6
Technical description	6-7
Installation and use	7-8

II - MAINTENANCE AND STERILIZATION

Suprasson handpiece maintenance	8-9
Tip #1 maintenance	9
Micromotor maintenance	9
Unit maintenance	10
Fuse replacement	10
Trouble shooting	11-12

III - LIABILITY 13

IV - SYMBOLS 13

I - PRESENTATION AND INSTALLATION

1. 1 PRESENTATION

COCOON and COCOON SPRAY are devices specially adapted for the veterinary practice. It includes a micromotor and a multi-function ultrasound generator.

Device are equipped with :

- 1 green flexible tube (2 m length) for connection to a water source, fitted with an impurity filter.
- 1 detachable mains cord + earth (2 m length).
- 1 single disconnectable pedal
- 1 AV40 micro motor
- 1 sterilisable ultrasonic handpiece
- 1 tip #1
- 1 handpiece support (Fig. 1-9)
- 1 micromotor support (Fig. 1-8)
- 1 water pump (available on option)

1. 2 TECHNICAL DESCRIPTION

a) Front panel (Fig. 1)

- ON/OFF switch (I/O) (Fig. 1-1).
- Power adjustment (Fig. 1-2).
- Range selection (motor/endo/scaling) (Fig. 1-3).
- Spray adjustment (Fig. 1-4).
- Motor rotation reversal button (Fig. 1-5)
- Ultrasonic handpiece cord (Fig. 1-6)
- Micro-motor cord (Fig. 1-7)

b) Rear panel (Fig. 2)

- Pedal power socket (Fig. 2-1).
- Mains socket (Fig. 2-2).
- Fuses housing (Fig. 2-3).
- Flexible hose for connection to a water source (Fig. 2-4).

c) Technical Data

Power supply : 230 VAC, 50/60 Hz

Class I equipment, BF Type

Fuses : 2 x 0.5 AT

Vibration frequency : 27 to 32 KHz

Water pressure : 1 to 5 bar (14.5 to 72.5 PSI)

Water filter : about 60 µm

Motor : 40000 rpm

Overall dimensions :

Depth : 180 mm Width : 230 mm

Height : 100 mm Weight : 3 Kg

Manufactured in accordance with Directive 89/336/EEC on electromagnetic compatibility.

1. 3 INSTALLATION AND USE

a) Recommendations

Before the unit is used for the first time, connect the accessories and operate the spray for a few minutes to rinse the water circuits inside the instrument.

b) Connections

Rear panel of COCOON / COCOON SPRAY :

Connect :

- The green water inlet tube (Fig. 2-4) to the water supply source or to the pump (depending on option).
- The mains power cable must be plugged into its socket (Fig. 2.2) and to a socket connected to earth (according to country).

c) Commissioning and use

Motor support installation

Observe correct direction of assembly of the micromotor support; if reversed, there is a risk of breakage (Fig. 3).

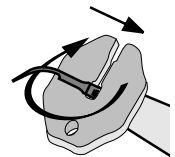
Suprasson handpiece connection

Connect the handpiece by plugging it into the cable socket.

Do not rotate.

Tip mounting

The Suprasson handpiece is a high technology instrument requiring precautions for use. The tip must be screwed and moderately tightened by means of the corresponding wrench.

CAUTION : Correct tip vibration requires perfect tightening without overtightening beyond the thrust point. Remove the tip after each procedure to avoid self-jamming.

Scaling with Tip #1

Interdental treatment (Fig. 4A)

Tooth neck and subgingival treatment (Fig. 4B)

Tangential application

- Do not apply the tip extremity directly onto the tooth.
- Apply the tip firmly without pressure.
- The Suprasson handpiece reciprocates slowly and evenly.

Suprasson COCOON / COCOON SPRAY adjustment

Average values for indication only

Tip #1	Min. power : 1
	Max. power : 10

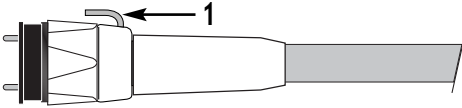
Micromotor connection

Connecting the micromotor to its lead is done by simply screwing it in.

The two red reference marks must be lined up.

COCOON SPRAY

Attach the tube delivered with the contra-angle to the small L-shaped outlet connector on the micromotor (1).

Spray adjustment is achieved via the front panel (Fig. 1-4).

In the case of prophylaxis contra-angle utilisation, close the front spray valve in order to stop the flow of irrigation.

On the front panel there is a three position button (Fig. 1-5) for reversal of the motor rotation :

Left selector : clockwise rotation

Right selector : anti-clockwise rotation

Centre selector : no rotation

II - MAINTENANCE AND STERILISATION

The following maintenance and/or sterilization instructions must be performed prior to any initial use of the handpiece or of the tip provided with the instrument.

2. 1 SUPRASSON HANDPIECE MAINTENANCE

a) **Maintenance of the Suprasson handpiece** (Fig. 5)

Handpiece body must be cleaned and disinfected (e.g. with alcohol : 96% V/V ethanol or dental disinfecting towelettes). The threaded cavity can be cleaned by means of a cotton swab soaked in alcohol. The handpiece must not be cleaned in an ultrasonic bath.

Caution : DO NOT disconnect the handpiece while the instrument is powered and the control pedal is activated.

b) **Sterilization of the Suprasson handpiece**

Sterilization conditions and cycles according to equipment:

Equipment : autoclave

Wet heat sterilization:

134°C under 2 bar (200 kPa) for 20 min.

- Detach the handpiece from its connecting cord and remove the tip.
- Wrap the handpiece in a sterilizable cloth or pouch, as required.
- Remove the handpiece from the autoclave as soon as sterilization is complete.
- Carefully dry the electrical contacts of the handpiece and cable connector before connecting them.

Caution :

- Prior to sterilization, use compressed air to remove any irrigation fluid left in the Suprasson handpiece.
- Always sterilize the Suprasson handpiece without the tip installed.
- Because of use and successive sterilization procedures, check the Suprasson handpiece for any sign of external damage before you use it.
- The handpiece has an irrigation circuit gasket ring in its rear part (1.15 x1) accessible and visible thanks to a groove in the rear casing. As a result, the gasket ring can be easily replaced if it is damaged or excessively worn. Have it replaced or replace it yourself (Ref.: F12304: 3 water circuit gasket rings; F00070: ring fitting tool). With subsequent disconnections/re-connections and/or sterilisations, this gasket ring should be lubricated with dental instrument grease in order to prolong its efficiency and prevent internal leaks.

2. 2 TIP #1 MAINTENANT

a) When is a scaling tip worn out ?

The shape and mass of the tip are determining characteristics for achieving maximum ultrasonic generator efficiency. Both characteristics should be monitored in order to maintain the best possible performance of the unit.

Therefore, it is strongly advised not to alter the structure of a #1 tip by filing or bending.

Similarly, the ageing of tips simply due to normal wear results in a change in their characteristics.

Tips damaged as a consequence of wear or accidental impacts (fall, etc.) must be systematically replaced.

b) Sterilization of #1 tip

All tips must be cleaned by using a cotton swab soaked in alcohol or towelettes or by immersion in an ultrasonic bath.

Tips can be sterilized by any conventional thermal method.

Sterilization conditions and cycles according to equipment :

Equipment: autoclave

Wet heat sterilization :

134°C under 2 bar (200 kPa) for 20 min.

When sterilizing handpiece and #1 tip, avoid any contact between metallic parts of a different nature as this would create electrolytic couples liable to result in localized deterioration. To avoid this phenomenon, wrap the items in.

2. 3 MICRO-MOTOR MAINTENANCE

Cleaning and disinfecting

Caution : Do not disconnect the micro-motor while the device is switched on and the control pedal is engaged.

The micro-motor body must be cleaned and disinfected (e.g. with alcohol: 96% V/V ethanol and/or dental disinfecting towelettes) without immersion.

The micro-motor and its connecting cable must never be immersed in a solvent or in an ultrasonic cleaning bath.

2. 4 UNIT MAINTENANCE

The COCOON / COCOON SPRAY case and the Suprasson handpiece cord and micromotor cord can be cleaned and disinfected with alcohol and/or disinfecting towelettes as used in dental practice (Fig. 6).

2. 5 FUSE REPLACEMENT

- Unplug the mains cord from the power outlet (Fig. 7-1).
- Insert a screwdriver end into the slot under the three male pins of the mains socket (Fig. 7-3).
- Pull out and check the defective fuse(s).
- Replace the defective fuses as required with fuses of the same type and rating (see page 7).
- Push the fuse holder back into its housing (Fig. 7-2).

2. 6 TROUBLESHOOTING

FAULT	POSSIBLE CAUSE	REMEDY
No operation	<ul style="list-style-type: none"> • Wrong mains cord connection • Blown fuse • Presence of fluid or moisture between handpiece and cord 	<ul style="list-style-type: none"> • Check mains outlet • Return to customer service • Dry any moisture, especially on electrical contacts
No spray	<ul style="list-style-type: none"> • Wrong water tube connection • No water pressure • Clogged filter or defective solenoid valve 	<ul style="list-style-type: none"> • Check water supply • Check pressure • Clean or replace filter • Return to customer service
No water at tip but vibration present	<ul style="list-style-type: none"> • Blocked tip • Incorrect spray adjustment 	<ul style="list-style-type: none"> • Remove obstruction • Adjust spray
Insufficient power Low vibration	<ul style="list-style-type: none"> • Worn or deformed tip • Incorrect use: wrong angle of application or inadequate pressure on tooth • Presence of fluid or moisture between handpiece and cord 	<ul style="list-style-type: none"> • Replace tip • See tip utilization method • Wipe electrical contacts dry
No ultrasonic output	<ul style="list-style-type: none"> • Incorrect tip tightening • Defective connector contact • Cut wire in handpiece cord 	<ul style="list-style-type: none"> • Tighten with wrench • Clean cord contacts • Return to customer service for cord replacement
Water leak between handpiece and cord or at front/back junction of handpiece	<ul style="list-style-type: none"> • Worn sealing gasket (1.15 x 1) in handpiece 	<ul style="list-style-type: none"> • Replace seal

FAULT	POSSIBLE CAUSE	REMEDY
Motor will not run after the pedal has been activated	<ul style="list-style-type: none"> • Wrong pedal cable connection • Motor rotation reversal button in central position • Wrong motor cable connection • Motor cable jack loose on cable connector • Cable connector contact blades too flattened • Control box malfunction • Cut cable • Defective pedal • Motor failure 	<ul style="list-style-type: none"> • Plug in pedal cable jack correctly • Turn it to desired position • Plug in motor cable jack correctly • Tighten motor cable jack • Slightly raise the 2 blades by means of a screwdriver • Return to SATELEC Customer Service • Change cable • Change pedal • Change motor
No water output at contra-angle	<ul style="list-style-type: none"> • Water tube incorrectly connected • Insufficient water pressure • Blocked filter of faulty electrovanne • Contra-angle blocked • Micromotor water outlet connector blocked 	<ul style="list-style-type: none"> • Check water input • Check pressure • Clean or change filter • Return to SATELEC Customer Service • Remove obstruction • Remove obstruction
The luminous switch does not light up	<ul style="list-style-type: none"> • Fuses blown 	<ul style="list-style-type: none"> • Replace fuses
Rotation direction does not match selection	<ul style="list-style-type: none"> • Incorrect micromotor connection 	<ul style="list-style-type: none"> • Check micromotor for correct assembly onto connecting cable (red mark)

III - LIABILITY

The manufacturer's liability shall not be pledged in the following cases:

- Failure to respect the manufacturer's instructions for installation (water pressure, mains voltage).
- Maintenance or repair by persons not authorized by the manufacturer.
- Use on an electrical installation that does not comply with the regulations in force
- Other uses as those specified in this Manual.
- The manufacturer reserves the right to modify the device and/or the instruction manual without prior notice.

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S/N°

Made in France by
SATELEC
Z.I. du Phare 33708 MERIGNAC FRANCE

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S/N°

Made in France by
SATELEC
Z.I. du phare 33708 MERIGNAC FRANCE

 Alternating current

 BF Type

 Caution, refer to accompanying documents

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