

EICKEMEYER[®] LED Wood's Lamp

Item no. 179030

Scope of application

The product is used to increase the operator's viewing angle and facilitate the observation of object details.

Precautions

1. After the instrument is used, please put it in the instrument box and store it in a ventilated and dry place.
2. When there is dust on the lens surface, please gently brush it off with a soft brush or gently wipe it with lens paper. Do not wipe with hard paper or unclean objects, and avoid ethanol-based, amino- or acetone-based cleaners. Never immerse a medical magnifying glass in liquid.
3. Do not use in a high temperature, humidity, full of water vapor or direct sunlight environment.
4. Do not disassemble the medical magnifying glass without authorization.
5. Do not look directly at the light source.
6. If disinfection is required, please take a small amount of medical grade chloric acid disinfectant with medical absorbent cotton under the condition that the power key of medical magnifying glass is off, and gently wipe the surface of the shell of medical magnifying glass for disinfection.

Usage

1. Turn on the power button on the handle of the medical magnifying glass and light the ring light.
2. Place a medical magnifying glass close to the examined skin to observe the skin lesions, color and fluorescence reaction of vitiligo, psoriasis, acne and other skin diseases.
3. After use, press the power button to turn off the power.

Storage method

This product should be wiped clean and put into the instrument box, stored in the relative humidity is not more than 80 %, no corrosive gas, well ventilated, dry and cool room.