

EickCut

30495013

The cutting forceps was delivered in parts, how do I assemble it?

You can find an assembly instructions video on:
https://youtu.be/cw-3run3_4w
(click on camera symbol)

How long does a coagulation take?

Until the tissue turns white, about 2–3 seconds. If the tissue turns black, it was too long.

How often can the EickCut be processed?

Handle and metal parts are designed for 100 reprocessing cycles. The electrodes should be replaced after 50 cycles.

Die Zange kam in Einzelteilen, wie baue ich sie zusammen?

Sie finden eine Video-Aufbauanleitung unter:
https://youtu.be/fXWy7OkA_Sw
(Klick auf Kamera-Symbol)

Wie lange muss koaguliert werden?

Bis das Gewebe weiß wird, ca. 2–3 Sekunden. Wird das Gewebe schwarz, war es zu lange.

Wie oft darf die EickCut aufbereitet werden?

Griff und Metallteile sind für 100 Aufbereitungszyklen ausgelegt. Die Elektroden sollten nach 50 Zyklen ausgetauscht werden.

EICKEMEYER®

GERMANY | Eltstraße 8 | 78532 Tuttlingen | Deutschland | T +49 7461 96 580 0 | F +49 7461 96 580 90 | info@eickemeyer.de | www.eickemeyer.de

SWITZERLAND | Sandgrube 29 | 9050 Appenzell | Schweiz | T 071 788 23 13 | F 071 788 23 14 | info@eickemeyer.ch | www.eickemeyer.ch

UNITED KINGDOM | 3 Windmill Business Village | Brooklands Close | Sunbury-on-Thames | Surrey | TW16 7DY | UK | T 020 8891 2007 | www.eickemeyer.co.uk

DENMARK | Solbakken 26 | Hammelev | 6500 Vojens | Danmark | T +45 70 20 50 19 | info@eickemeyer.dk | www.eickemeyer.dk

NETHERLANDS | Bellweg 44 | 4104 BJ Culemborg | Nederland | T +31 345 58 9400 | info@eickemeyer.nl | www.eickemeyer.nl

ITALY | Via G. Verdi 8 | 65015 Montesilvano (PE) | Italia | T 085 935 4078 | F 085 935 9471 | info@eickemeyer.it | www.eickemeyer.it

POLAND | Al. Jana Pawła II 27 | 00-867 Warszawa | Polska | T 22 185 55 76 | F 22 185 59 40 | info@eickemeyer.pl | www.eickemeyer.pl

CANADA | 617 Douro Street, Suite #205 | Stratford | Ont. | Canada | N5A 0B5 | T +1 519 273 5558 | F +1 519 271 7114 | info@eickemeyervet.ca | www.eickemeyercanada.ca