


LifeVet® 10 C Multiparameter Monitor


Item no. 321920

Network Connection & Remote View

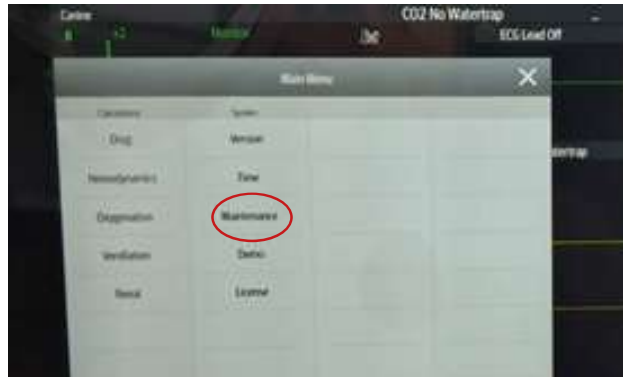

1. Main menu on the patient monitor


- ▶ Select Main Menu --> System --> Maintenance
- ▶ The system password is: 888888

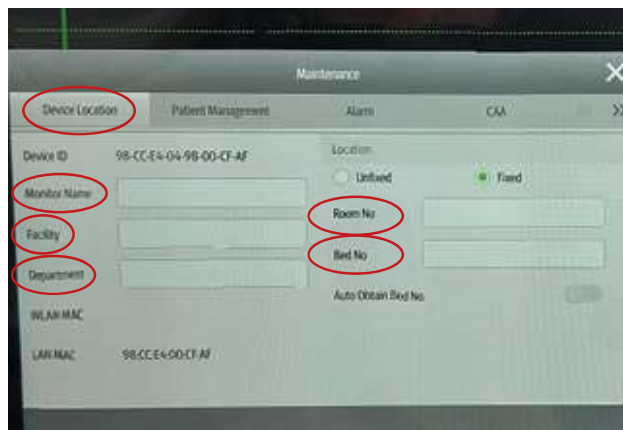

2. Patient monitor

Set the following information on the device you want to monitor:

- ▶ Monitor name
- ▶ Facility
- ▶ Department
- ▶ Room no.
- ▶ Bed no.

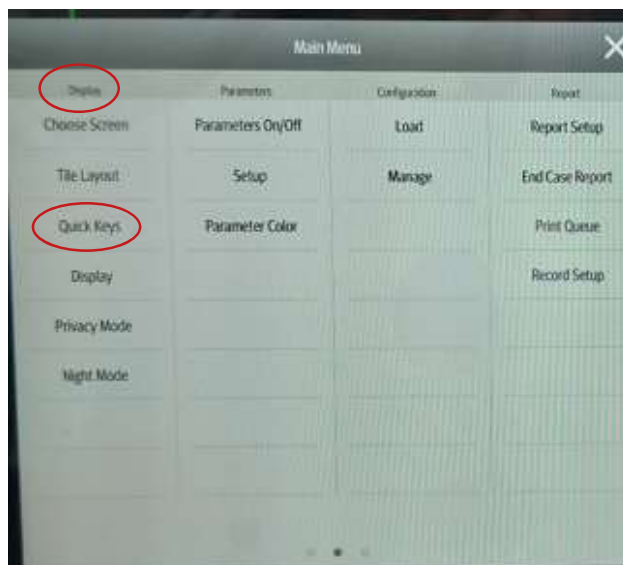


3. Monitoring monitor

Select Main Menu --> Display --> Quick Keys

LifeVet® 10 C Multiparameter Monitor


Item no. 321920

Network Connection & Remote View

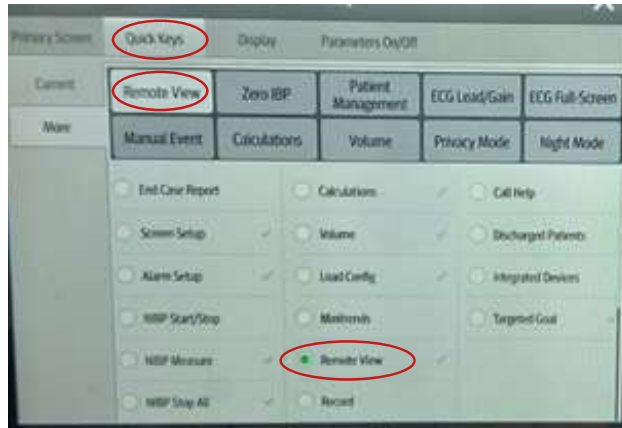

4. Monitoring monitor

- Select a Quick Key that you do not need and replace it with Remote Display Off

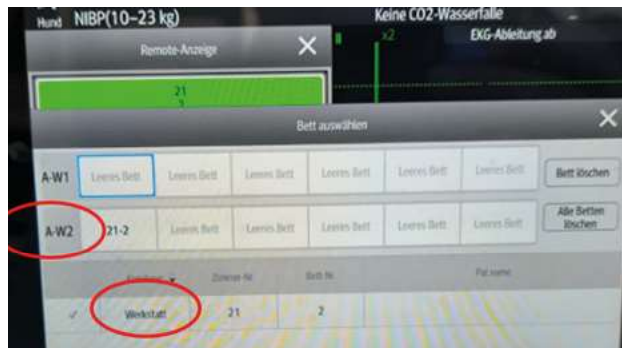

- Close all settings and open Remote Display
- The table below shows all patient monitors connected to the network
- Select the monitors to be displayed and add them to the W list

LifeVet 10C

Left: Patient monitor

Right: Monitoring monitor

