

CAWO Imaging Plate Cleaner



A dedicated cleaner for CR imaging plates. Protect your investment...



CAWO Imaging Plate Cleaner

is an easy-to-use, ethanol based cleaner that ensures longer life and higher performance for CR imaging plates.

TECHNICAL SPECIFICATIONS

Solvent

- Ethanol 99,7%

Additives

- Catanac SN (Cyastat SN 50)
0,3 %

Contains

- 0,5 l

Safety

- Highly flammable
- Keep container closed
- Keep away from sources of ignition

Safety datasheets are available on request

CAWO Photochemisches Werk GmbH

P.O. Box 1129

D - 86521 Schrobenhausen

Phone: (++49) 8252 9109-0

Fax: (++49) 8252 9109-19

Internet: www.cawo.com

E-mail: cawo@cawo.com

Protect your investment

Dust and dirt build up on CR phosphor plates, leading to image artifacts. To avoid this, plates need cleaning at regular intervals. As CR imaging plates are highly sensitive to water-based cleaning fluids, CAWO has developed an ethanol-based plate cleaner. Using this cleaner, specially designed for CR phosphor plates, will help ensure them the longest service life.

Simple to use

CAWO Imaging Plate Cleaner is simple to use: moisten a piece of gauze with the cleaner, wipe gently across the plate's surface and wait for any remaining solvent to evaporate.

CAWO Imaging Plate Cleaner is suitable for use with CR imaging plates only, not with conventional intensifying screens.

Cleaning frequency

Plates subject to a normal frequency of use should be cleaned at least once every month or whenever there is doubt about particles visible on the X-ray images.

Characteristics of the products described in this publication can be changed at any time without notice.

