INSTRUCTIONS FOR USE OF:

iM3[®] 42-12 ULTRASONIC SCALER

FOR VETERINARY USE ONLY

ENGLISH MANUAL

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ČESKÝ MANUÁL

www.im3vet.com

1. INTRODUCTION

ABOUT THIS DOCUMENT

Please read this manual carefully and explore all the equipment's extraordinary features:

- ✓ Please see **Introduction Section** for a list of symbols used.
- ✓ **Section 2** is an overall description of what makes iM3's 42-12 Scaler so unique.
- ✓ **Section 3** contains a complete list of accessories, detachable parts and consumables.
- ✓ The unit shall be installed as described in **Section 4**.
- ✓ **Section 5** describes how to use the equipment.
- ✓ **Section 6** is the trouble shooting guide.
- ✓ Read details about infection control and cleaning in **Section 7**.
- ✓ **Section 8** details for the Service and Repair of 42-12 Ultrasonic scaler.
- ✓ **Section 9** provides technical product specifications and manufacturer details.
- ✓ **Section 10** contains Legal information for your clarification.

For more information, please contact customer service in your country :

EU & UK: support@im3dental.com | US: info@im3usa.com | AU: support@im3vet.com

or you can contact your local distributor.

These installation and operating instructions form part of the unit. Failure to comply with the instructions and information in these installation and operating instructions means that iM3 will not be able to offer any warranty or assume any liability for the safe operation and the safe functioning of the unit.

WARNING! No unauthorized installation, service or modification of this equipment is allowed. Unauthorised modification could result in a malfunction or physical injury!

DEFINITION OF SYMBOLS

Please see below a list of symbols which can be found on:

- » Instructions for use manual » the device/unit » the packaging

WARNINGS SYMBOLS

The warnings in this document are intended to draw your attention to possible injury to persons or damage to machinery.

The warnings are structured as follows:

SIGNAL WORD: Description of the type and source of danger

The possible consequences of ignoring the warning are described here.

Follow these measures to avoid the danger.

The signal word differentiates between four levels of danger:

- » **DANGER**
Immediate danger of severe injury or death
- » **WARNING**
Possible danger of severe injury or death
- » **CAUTION**
Risk of minor injuries
- » **NOTICE**
Risk of extensive material/property damage

The following warning symbols are used:

	WARNING: General Warning Symbol
	WARNING: Risk of injury
	ATTENTION : something could be damaged.
	PROTECTION : Bio-Hazard warning
	PROTECTION : Wear protective gloves
	PROTECTION : Wear protective goggles
	PROTECTION : Use mouth & nose protection
	PROTECTION : Wear protective clothing

These installation and operating instructions form part of the unit. Failure to comply with the instructions and information in these installation and operating instructions means that iM3 will not be able to offer any warranty or assume any liability for the safe operation and the safe functioning of the unit.

OTHER SYMBOLS USED

	General information		Important information
	Consult Instructions for Use		Manufacturer
	Date of Manufacture		Expiry Date
	Product Code		Serial Number
	Medical Device		Data structure in accordance with Health Industry Bar Code
	CE labelling: compliance with EU legislation		UKCA: UK Conformity Assessed for products sold within Great Britain.
	UL Component Recognition Mark indicates compliance with Canadian and U.S. requirements		Data Matrix Code for product information including Unique Device Identification
	Caution! According to Federal law, this medical device may only be sold by or on the order of a dentist, physician or any other medical practitioner licensed by the law of the state in which he or she practices and intends to use or order the use of this medical device.		
	Sterilizable up to the stated temperature		Thermo washer disinfectable
	Humidity limitation		Electrostatic sensitive devices
	Permitted temperature range		Supply voltage of the device
	Type B applied part (not suitable for intracardiac application)		Suitable for people with pacemakers or implanted defibrillators
	Fragile, handle with care		This way up
	DO NOT dispose of with domestic waste	Thread system: Q-Link Ultra style connection P6 style connection	

These installation and operating instructions form part of the unit. Failure to comply with the instructions and information in these installation and operating instructions means that iM3 will not be able to offer any warranty or assume any liability for the safe operation and the safe functioning of the unit.

RESPONSIBILITY OF THE MANUFACTURER

The manufacturer can only accept responsibility for the performance of the device when it is used in compliance with the following directions:

- » The device must be used in accordance with these **Instructions for Use**.
- » Modifications or repairs must **ONLY** be undertaken by iM3 or an authorised iM3 service partner.
- » The device has no components that can be repaired by the user.
- » The electrical installation at the premises must comply with the regulations laid out in **IEC 60364-7-710 (Installation of electrical equipment in rooms used for approved purposes)** or with the regulations applicable in your country.
- » Unauthorised opening of the device invalidates all claims under warranty and any other claims.
- » Improper use, unauthorised assembly, modification or repair to the device, non-compliance with our instructions or the use of accessories and spare parts which are not approved by iM3, invalidates all claims under warranty and any other claims.
- » The appliance is designed to be used by a qualified Veterinary professional **ONLY**. The appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- » Children should be supervised to ensure that they do **NOT** play with the appliance.
- » **FOR VETERINARY DENTAL USE ONLY.**

EXPLANATION OF WARRANTY TERMS

- » **This iM3 device has been manufactured with great care by a qualified team member. A wide variety of tests and controls ensures faultless operation. Please note claims under warranty can only be validated when all the directions in the instructions for use have been followed.**
- » As the manufacturer, iM3 is liable for material or manufacturing defects within the 12 months from the date of purchase. This does not include broken stacks (inserts), or broken handpiece tips.
- » Freight to and from Repair Centre will be the Owner's responsibility.
- » We accept no responsibility for damage caused by incorrect handling or by repairs carried out by third party not authorised to do so by iM3.
- » Claims under warranty accompanied by proof of purchase, must be sent to the vendor or to an authorised iM3 service partner.
- » The provision of service under warranty extends neither the warranty period nor any guarantee period, repair and returns.
- » In the event of operating malfunctions immediately contact an authorised iM3 service partner.
- » Repairs and maintenance work must only be undertaken by an authorised iM3 service partner.

For a list of iM3 Authorised Service Partners in your country please visit:

Europe & UK: www.im3vet.eu

USA: www.im3vet.com

Australia/Asia: www.im3vet.com.au

COPYRIGHT INFORMATION

All circuits, processes, names, software programs and units mentioned in this document are protected by copyright. The installation and operating instructions may not be copied or reprinted, either in full or in part, without written authorisation from iM3.

CONTENTS

1. INTRODUCTION	2	6. TROUBLE SHOOTING & MAINTENANCE.....	17
» About this document	2		
» Definition of Symbols	3	7. CLEANING & STERILISING	18
› Warnings symbols.....	3		
› Other symbols used	4	8. SERVICE & REPAIR.....	19
» Responsibility of the Manufacturer	5		
» Explanation of Warranty Terms	5	9. TECHNICAL PRODUCT DETAILS.....	20
» Copyright information.....	5		
2. UNIQUE DESIGN FEATURES.....	7	10. LEGAL INFORMATION	21
3. 42-12 SCALER UNIT COMPONENTS	8	» Manufacturers Liability	21
» Control Centre Main elements	8	» Unauthorized Interference	21
» Control Centre back elements.....	9	» Claims Under Warranty.....	21
» Components of the iM3 42-12 Scaler Unit:	10	» Transport For Repair.....	21
› Optional Extra:.....	10	» Copyright Information.....	21
4. INSTALLATION	11		
5. OPERATION	12		
» Operating Technique.....	13		
» Life expectancy of the Scaler Tip:	14		
» Handling & Care	14		
› The Scaler Tip Inserts	14		
» Changing Scaler Rod or Tips	15		
› Changing the Ferrit Rod	15		
› Changing Scaler Tips.....	15		
» Instrument Tips	16		

2. UNIQUE DESIGN FEATURES

The iM3 42-12 Ultrasonic Scaler is a unique Scaler as it uses high ultrasonic frequency (42,000Hz), using a circular tip movement (with a very small amplitude 0.01 – 0.02mm) to remove plaque and calculus. The unique Titanium Scaler tips have no sharp sides, is fitted with a ferrite rod and generates little or no heat so can be safely used subgingivally up to 12mm – hence the name 42-12.


iM3 42-12 Ferrite Rod and Scaler Tip Insert

1. Ferrite Rod/Wrench (For the iM3 42-12 Scaler Tip Insert)
SKU EU/AU: O6860 | US: O1292
2. Universal Tip (For supra and sub-gingival scaling)
SKU EU/AU: O6840 | US: O0809
3. Perio Tip (For supra and sub-gingival scaling - long)
SKU EU/AU: O6850 | US: O0806
4. Thinline straight tip (ideal for felines and small mouths)
SKU EU/AU: O6845 | US: O0802


"A survey of 4,737 AAHA Members in the USA - of which 667 responded completing a comparison of the five major ultrasonic dental scalers in fifteen categories - iM3 came in first in every category, scoring a 90% overall satisfaction."

TRENDS MAGAZINE
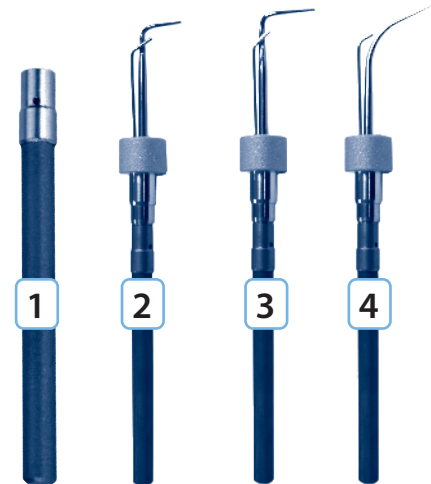


3. 42-12 SCALER UNIT COMPONENTS

CONTROL CENTRE MAIN ELEMENTS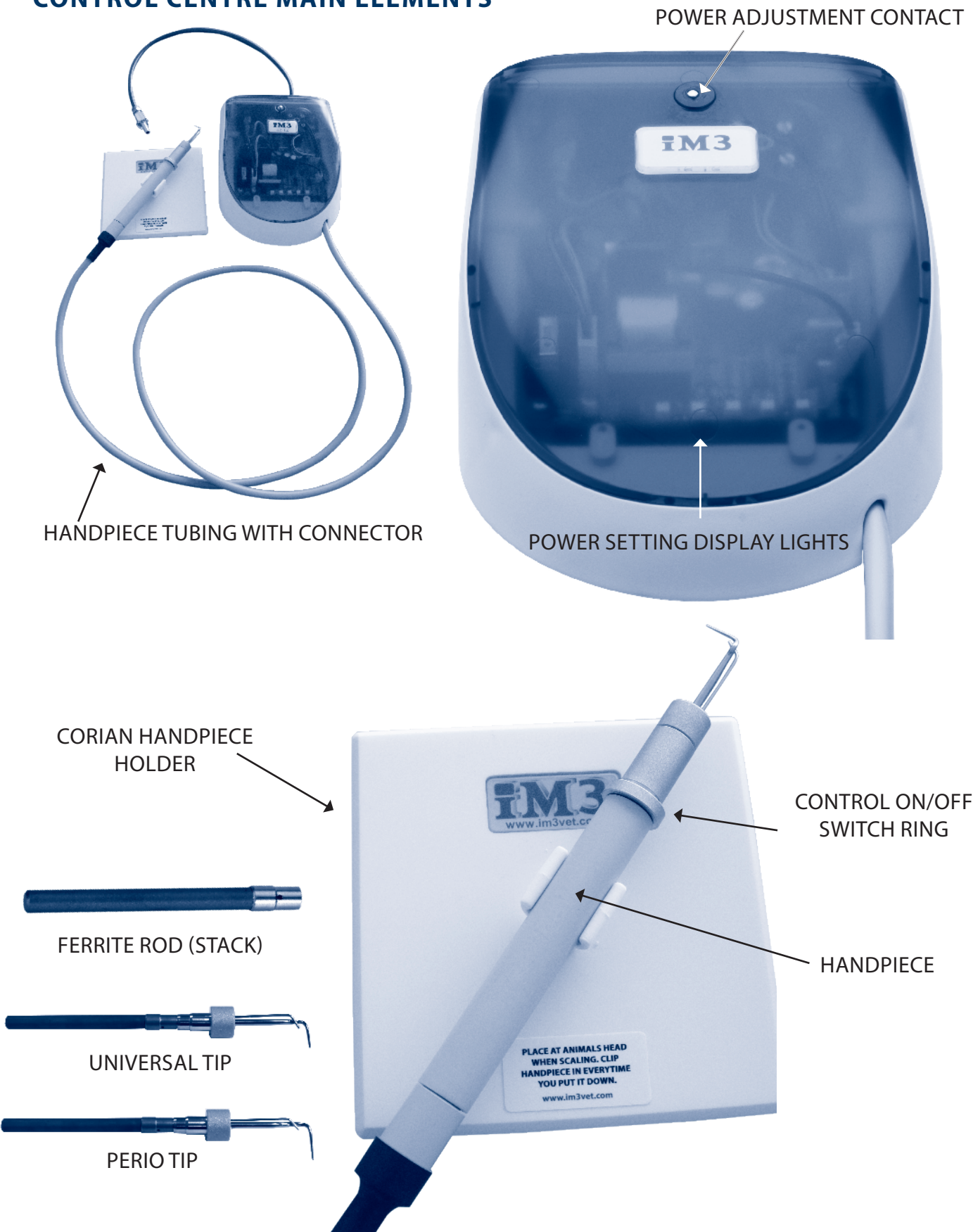

CONTROL CENTRE BACK ELEMENTS

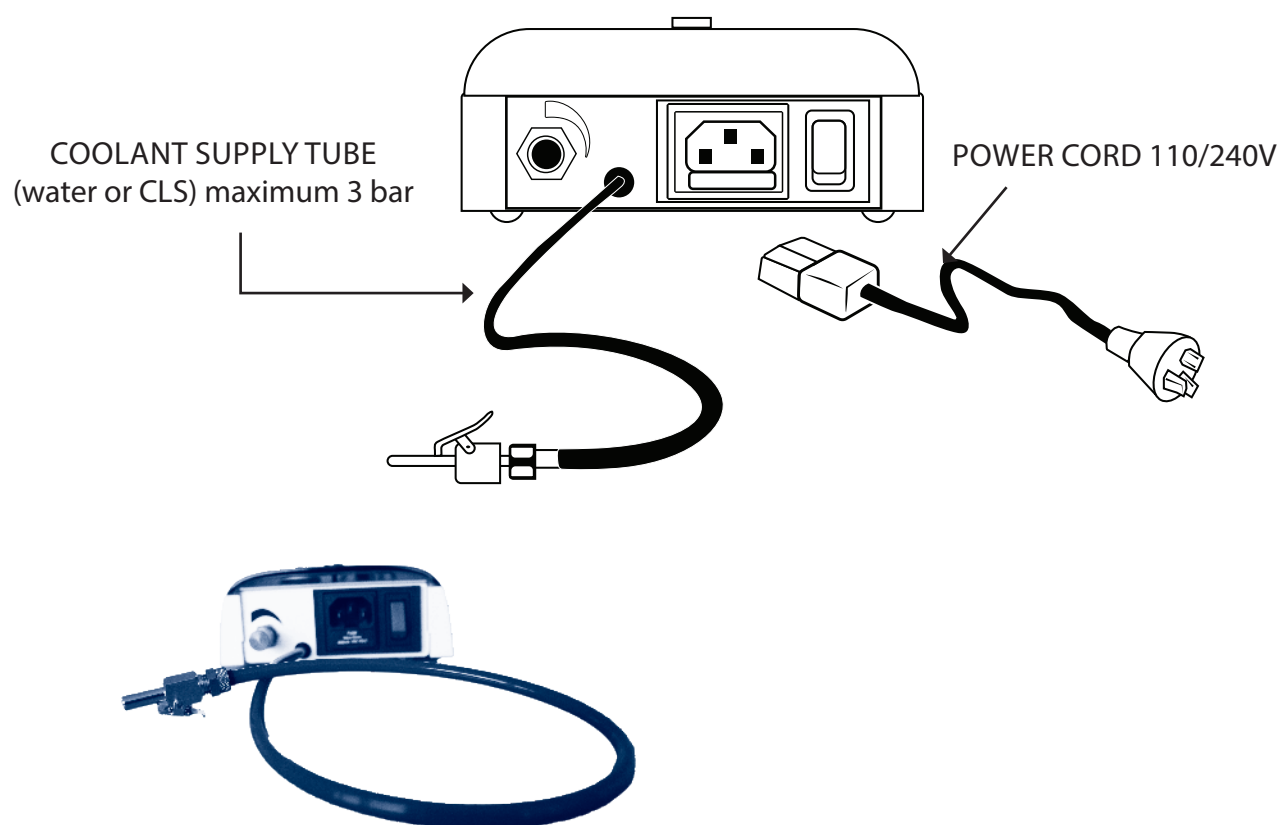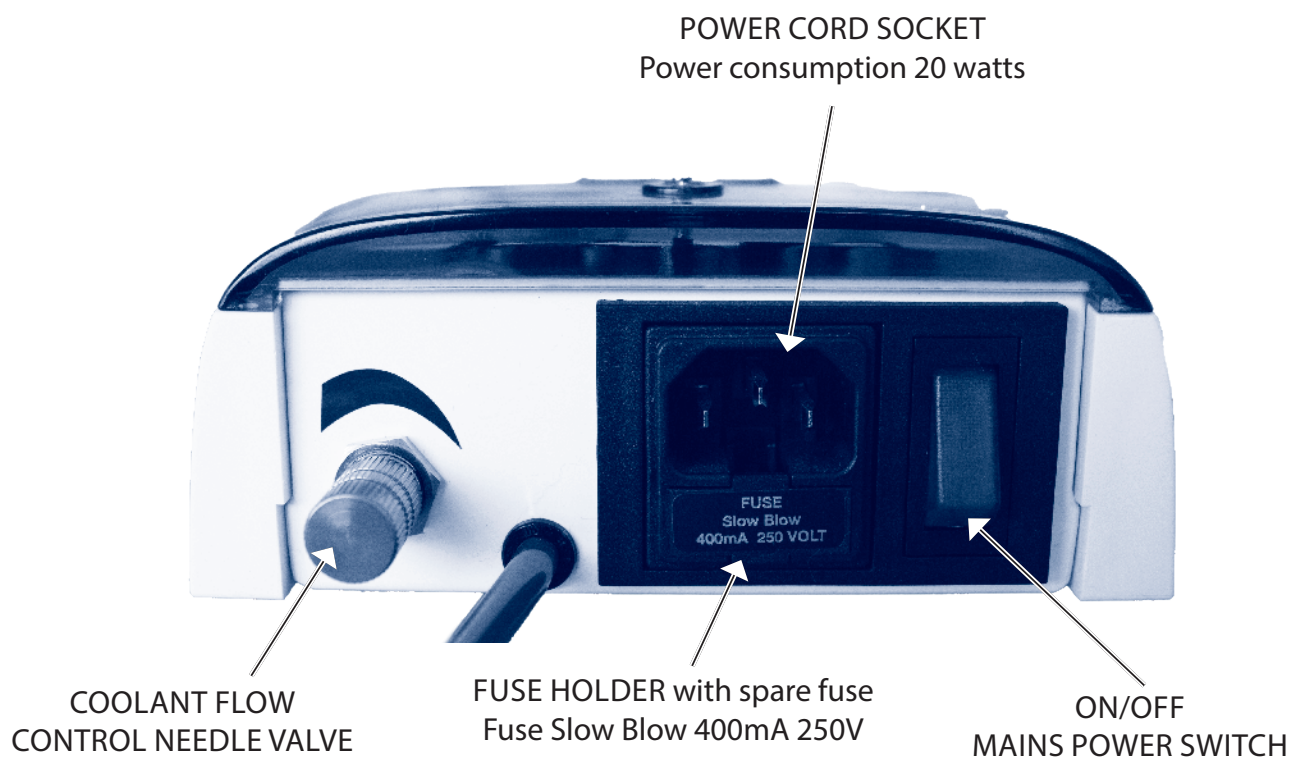COMPONENTS OF THE iM3 42-12 SCALER UNIT:

1. CONTROL CENTER
2. POWER ADJUSTMENT CONTACT
3. HANDPIECE TUBING with connector
- SKU EU/AU: O6950 | US: O4229
4. HANDPIECE - SKU EU/AU: O6900 | US: O1002
5. CONTROL ON/OFF SWITCH RING
6. TIP INSERT
7. FERRITE ROD (Stack) - SKU EU/AU: O6860 | US: O1291
8. ROD PLIERS - SKU EU/AU: O6882 | US: O1295
9. STEEL PIN - SKU EU/AU: O6880 | US: O1293
10. FERRITE ROD HOLDER (Wrench) - SKU EU/AU: O6865 | US: O1292
11. YELLOW CLAMP (Wrench) - SKU EU/AU: O1295 | US: O1295
12. CORIAN HANDPIECE HOLDER - SKU EU/AU: O6925 | US: O8000
13. COOLANT FLOW CONTROL NEEDLE VALVE
- SKU EU/AU: I6780 | US: P7056
14. COOLANT SUPPLY TUBE – (water or CLS),
maximum 3 bar - SKU EU/AU: O6930 | US: M7804
15. POWER CORD 110/240V - SKU EU/AU: O6910 | US: P6124
16. POWER CORD SOCKET, power consumption 20 watts
17. FUSE HOLDER with spare fuse – Fuse Slow Blow 400mA 250V
SKU EU/AU: O6891 | US: O2153
18. ON/OFF MAINS POWER SWITCH
19. POWER SETTING DISPLAY LIGHTS
20. COOLANT FILTER - SKU EU/AU: W0530 | US: P7235

FERRITE ROD HOLDER

YELLOW CLAMP WRENCH

STEEL PIN

FERRITE ROD

The iM3 42-12 comes with 2 tip inserts.

UNIVERSAL TITANIUM TIP
SKU EU/AU: O6840 | US: O0809

Designed for supra and sub-gingival scaling, approx 12mm (1/2") long.

PERIO TITANIUM TIP
SKU EU/AU: O6850 | US: O0806

Designed for supra and sub-gingival scaling.
The extra length (18mm (3/4") long
is ideal for deep pockets.

OPTIONAL EXTRA:

BEAVER TAIL TITANIUM TIP
SKU US: O0805 – Only available in US

This tip is used for gross debridement of calculus.


THINLINE STRAIGHT TITANIUM TIP
SKU EU/AU: O6845 | US: O0802


Ideal for felines and small mouths.
Designed for supra and sub-gingival scaling.


4. INSTALLATION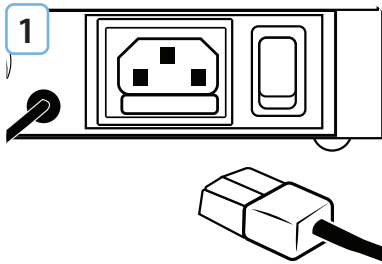


Insert the power cord into the rear of your unit. The unit automatically adjusts to either 110V or 240V AC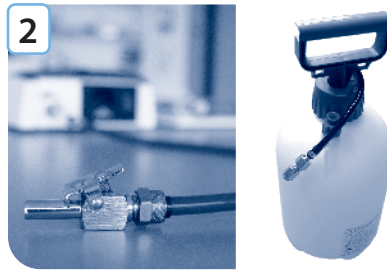

Connect the coolant supply tube to your coolant supply – Pressure should not exceed 3 BAR (40 psi). A pump up pressure tank is also available.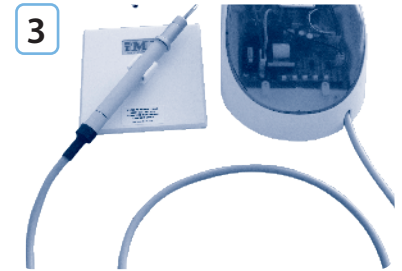

Connect the handpiece tubing) with the rear of the handpiece. Insert an instrument tip into the handpiece with a twisting motion until fully seated.

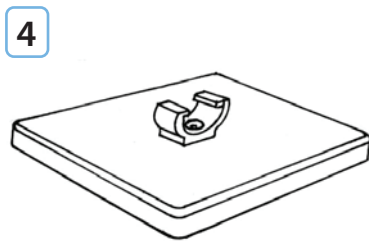
This holder is designed to reduce Stack breakage by preventing the Handpiece accidentally falling on the floor this usually occurs when the Handpiece is placed on the table while adjusting the animal's head.


Position the Corian Handpiece Holder right at the animals head, and make a habit of every time you put the Handpiece down, place it in the holder. It will help reduce accidental stack breakage


The iM3 2000 Professional Dental Unit has a handpiece holder on the front for the 42-12 Ultrasonic Scaler.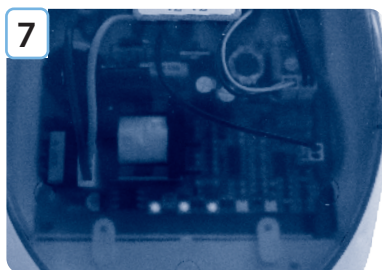

The Actual power setting can be seen through the Top of the Control Box. One light indicates lowest power and five lights indicate maximum power.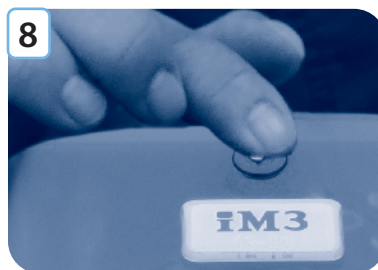

A touch on the power adjusting spot will alter the power setting.

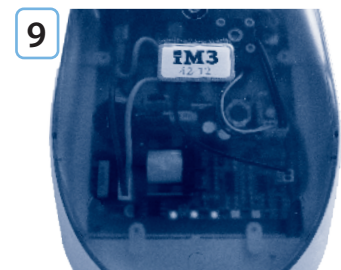One light (minimum power) is approximately 25% of maximum power. Normal scaling is 2 or 3 lights illuminated.

5. OPERATION

1. The 42-12 is turned ON or off at the rear of the Control Box. When ON, the switch will be illuminated red. The Power lights will also be illuminated.
2. The Handpiece tip is activated by sliding the silver control ring to the ON position (away from the instrument tip), causing the tip to vibrate and the coolant to flow. Make sure the adjusting knob on the coolant valve is turned anti-clockwise to allow the coolant to flow.
3. Tip Vibration – Power setting (Amplitude). Touching the power adjusting spot on the top of the Control Box will alter the amplitude (Power setting). Power adjustments should only be made with the handpiece control ring in the OFF position.
4. **Scaling in General:** The unique circular movement of the 42-12 tip provides a gentle, but highly efficient means of scaling a tooth. The circular motion with the high frequency and small amplitude, produces a very smooth result without hammering or scratching the tooth.





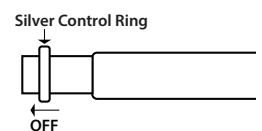
Fig.1- Circular motion of the 42-12 tip

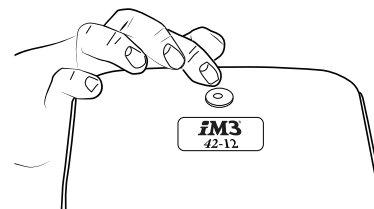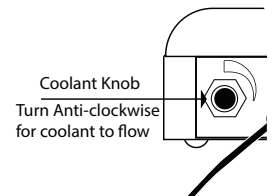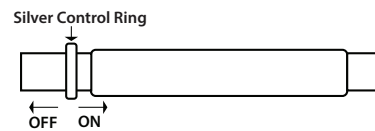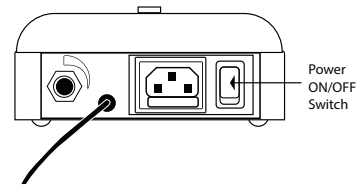OPERATING TECHNIQUE

It is important to use the 42-12 correctly to benefit from its unique features

1. Always use the tip parallel to the tooth (as pictured), with the side of the tip on the tooth.
2. Do not use the tip as a pick or chisel at right angles to the tooth, as this will scratch the enamel. Animal enamel is considerably thinner than human tooth enamel, and therefore more prone to damage.
3. Apply lightly to the tooth surface, sweeping back and forth as though one was painting the teeth with a paintbrush. Keep the tip moving. Use the tip to probe deep pockets, but always parallel to the tooth surface.
4. As the tip operates in a 360° circular motion, easy access is guaranteed, as all surfaces are active and there are no dead zones.
5. The circular motion actually brushes the teeth, rather than "hammering" them, and produces more of a polishing effect, rather than a scratching effect, as with other units.

NOTE: Increased pressure will not improve efficiency or the time taken to complete the procedure.

6. **Coolant flow control:** Adjust the spray to achieve a fine mist. The use of CLS is recommended in animals. Correct dilution is important (Read Manufacturer's Recommendations) as over dilution may cause the chlorhexadine to deposit small crystals.

NOTE: The use of any other solution other than CLS or distilled water in the 42-12 will void all warranties. CLS has been used successfully in air driven Dental Units and ultrasonic scalers since 1993

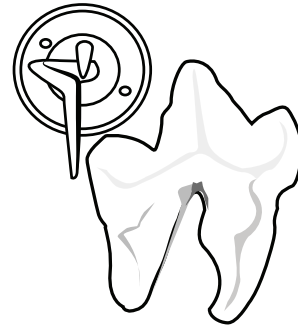
Fig.2 - Use tip parallel to the tooth

Fig.3- Keep tip moving in sweeping motion parallel to surface

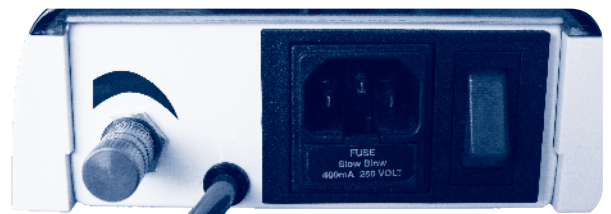
Fig.4- Coolant flow control located at back of scaler unit

Fig.5 - ONLY use CLS Amber bottle or distilled water (clear bottle) when using the 42-12 Scaler unit

LIFE EXPECTANCY OF THE SCALER TIP:

As the tip wears, due to use, the efficiency will be reduced. If performance is reduced, check that the Ferrite Rod (As pictured) is tight and that no foreign matter is in the handpiece. A loose ferrite rod is the most common cause of poor tip efficiency. Change the ferrite rod to see if an improvement can be gained – If no improvement is found, a new tip is indicated. Tip wear can be reduced by the use of CLS, as it contains glycerin, which will help lubricate the tip and reduce tip wear.

WARNING: Instrument tips that are bent, or scratched should be discarded immediately.

Do not think the “jack hammer” of other scalers improves efficiency. The 42-12 operates by using ultrasonic frequency, along with mechanical movement to remove and breakdown calculi.

The Ultrasonic sound will also destroy the cell wall of bacteria within a 1 to 2mm proximity, promoting improved dental mouth hygiene in your patients, particularly in deep sub-gingival pockets. The use of CLS with its chlorhexadine will further improve the disinfecting of such gingival pockets.

Fig.6 - Ferrite Rod to hold the iM3 42-12 Scaler Tip Inserts


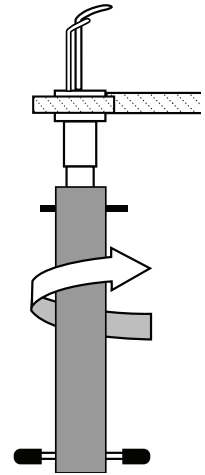
Fig.7 - Use tools to tighten rod firmly onto tip

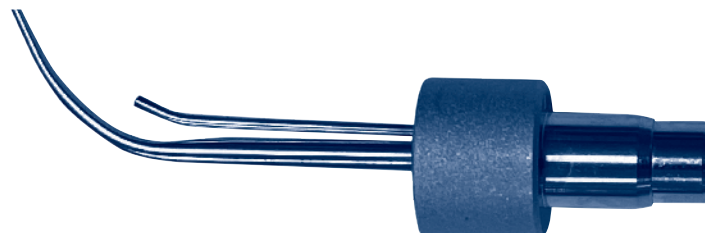HANDLING & CARE

THE SCALER TIP INSERTS

The shape of the tip is vital to the efficiency of the circular motion and ultrasonic effect.


WARNING:
DO NOT sharpen, bend or re-shape the tip.

Tip Wear Guide

Use the 42-12 Ultrasonic Scaler Tip Wear Guide to check whether your ultrasonic scaler tip needs to be replaced.

1. Tip wear will significantly reduce scaler performance as it will alter the ultrasonic frequency.
2. The ferrite rod must be tight – a loose or broken rod will cause poor or no vibration.
3. A correctly functioning tip used on a power setting of 3-4 lights will produce a coolant mist around the tip.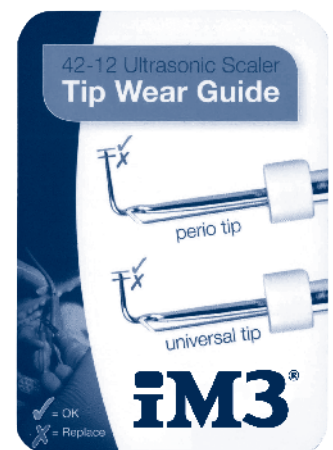


CHANGING SCALER ROD OR TIPS

CHANGING THE FERRIT ROD


Follow the below process to correctly remove and replace the Ferrit Rod:

1. Grip Collar of tip with yellow Clamp Wrench.
2. Slide Steel Pin through the hole in the Rod.
3. Slide blue Rod Tool over broken rod, steel pin engages with slot in top of tool. Grip rod with yellow clamp wrench and twist blue rod tool counter-clockwise to loosen ferrite rod.
4. Unscrew and remove the ferrite rod.
5. Screw new ferrit rod onto instrument tip.
6. Use tools (wrench and blue rod) to tighten new rod firmly onto tip.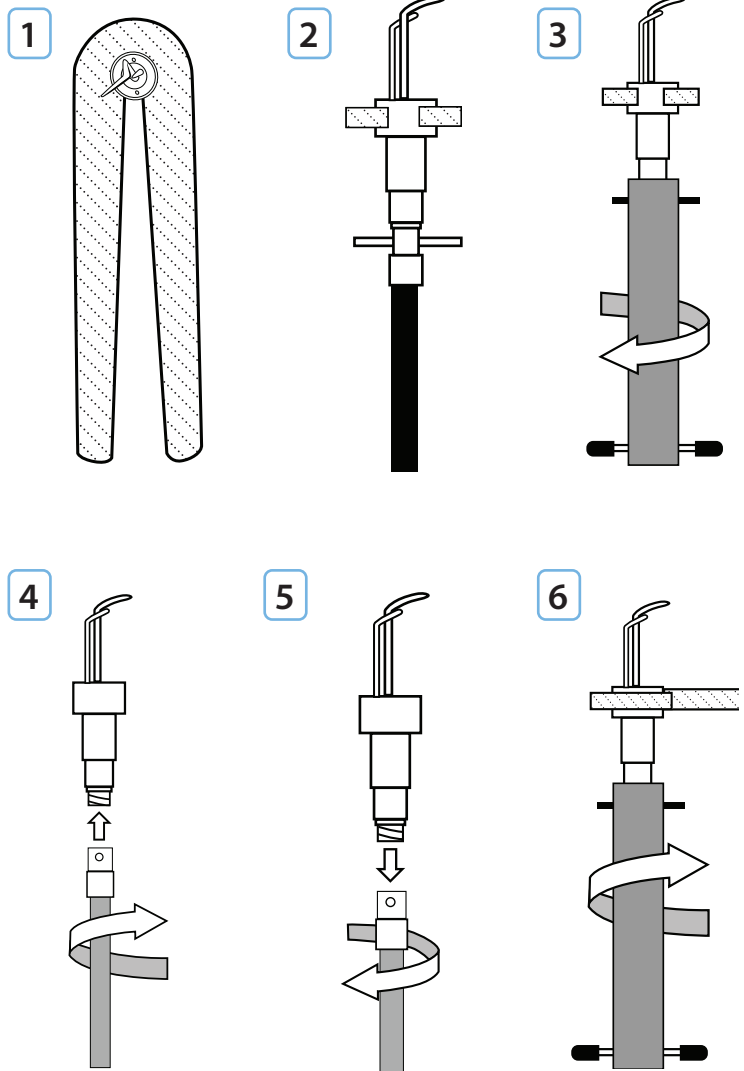

CHANGING SCALER TIPS

1. Hold the scaler tip by the metal collar, and twist out of the handpiece. A twisting motion is recommended as the best way of removing and inserting scaler tip inserts through the internal handpiece sealing O ring.
2. Shake the Handpiece to remove coolant liquid.
3. Insert new scaler tip insert with a twisting motion until fully seated.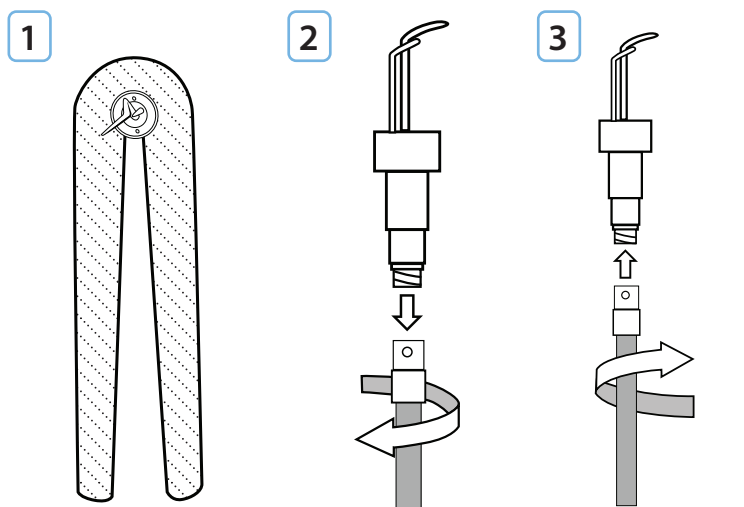

Grip Collar with
yellow clamp wrench


Unscrew and
remove the
ferrite rod.

Screw new
ferrite rod onto
instrument tip.

INSTRUMENT TIPS


All iM3 tips are made from titanium and are supplied complete with ferrite rod. The tips only fit the iM3 42-12 Ultrasonic scaler. All iM3 tips are Autoclavable.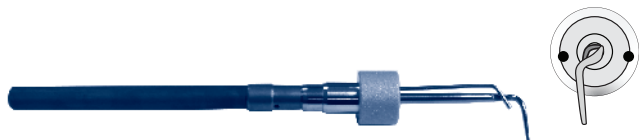

UNIVERSAL TITANIUM TIP

SKU EU/AU: **O6840** | US: **O0809**

For removal of Supra and Sub-gingival calculi in all areas. For general scaling with moderate calculi.

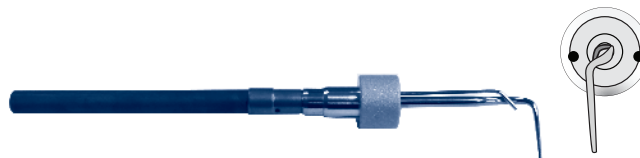PERIO TITANIUM TIP

SKU EU/AU: **O6850** | US: **O0806**

Removes Supra and sub-gingival calculi in all areas. Particularly useful in sub-gingival pockets, as deep as 13mm. The use of CLS will reduce the pathogens and pocket depth. Use the top in a similar fashion to a periodontal probe, and keep the tip moving with short strokes at all times.

BEAVER TAIL TITANIUM TIP

SKU US: **O0805** (Only available in US)

This tip is used for gross debridement of calculus.

THINLINE STRAIGHT TITANIUM TIP

SKU EU/AU: **O6845** | US **O0802**

Ideal for felines and small mouths. Designed for supra and sub-gingival scaling.

6. TROUBLE SHOOTING & MAINTENANCE

SYMPTOM	CAUSE	REMEDY	PART CODE
Coolant leaking between handpiece and connectors	Defective O Ring in Handpiece connector	Replace O Ring	O Ring: EU/ UK/AU: 06890 US: 04231
Coolant leak between Handpiece and scaler tip insert	Defective O Ring in Handpiece	Replace O Ring	O Ring: EU/ UK/AU: 06890 US: 04232
Coolant in interior of Handpiece	Defective O Ring in Handpiece	Replace O Ring	O Ring: EU/ UK/AU: 06890 US: 04233
Scaler tip vibrating, but no coolant spray	Coolant line blocked. Flow control valve closed	Check Filter - Open flow control valve	Water Filter EU/ UK/AU: W0530 US: P7235 Water Flow Control EU/ UK/AU: I6780 US: P7056
Poor or no tip vibration	Water in Handpiece	Empty Handpiece – if re-occurs, replace Handpiece.	Handpiece EU/ UK/: 06900 US: 01002
	Ferrite Rod loose	Tighten Ferrite Rod (See page 14)	N/A
	Broken Ferrite Rod	Replace Ferrite Rod (See page 15)	Ferrite Rod EU/ UK/: 06870 US: 01292
	Tip worn out	Replace tip	Universal Tip EU/ UK/: 06840 US: 00809 Perio Tip EU/ UK/: 06850 US: 00806
No Power	Blown fuse	Replace Fuse Slow Blow 400mA – 250 v	Slow Blow Fuse EU/ UK/: 06891 US: 02153

7. CLEANING & STERILISING

ITEM	METHOD	IMPORTANT
 SCALER TIP INSERT	Cleaned by chemical or steam autoclaving.	 Maximum: 180° C / 350°F.
 HANDPIECE	DO NOT Soak the Handpiece in liquid. Clean by wiping with alcohol, soap or water based disinfectant solutions.	 DO NOT USE GLUTARALDEHYDE, FORMALIN SOLUTIONS or any products containing aggressive chemicals.  Steam sterilize the Handpiece at a maximum temperature of 121°C / 250°F (Rubber Pack Cycle).  iM3 recommends autoclaving between patients.
 CONTROL BOX & CORD	Wipe with alcohol, soap or water based disinfectant solution.	 Solutions suitable for sanitizing telephones are generally satisfactory.

CAUTION: Avoid contact with temporary restorations, or porcelain crowns. Avoid contact with the tongue or mucosa with the un-cooled part of the tip.

8. SERVICE & REPAIR

Technical & Support

Call us anytime during business hours on our Tech Support Line and we'll take care of all your needs or email us with your query and we'll get back to you promptly

For servicing quotation contact us at:

Email: support@im3dental.com

Phone (Ireland): +353 (0) 1 691 1277

Phone (UK): +44 (0) 1 423 224297

or message us via our Website online form:
<https://im3vet.eu/servicing-eu/>

(USA): 1 800 664 6348 / info@im3usa.com

(AUS): +61 2 9420 5766 / support@im3vet.com

For a list of iM3 Authorised Service Partners in your country please visit:

Europe & UK: www.im3vet.eu

USA: www.im3vet.com

Australia/Asia: www.im3vet.com.au

DO NOT ✕ attempt to open the control box.

Service and repair can only be carried out by an approved iM3 Dealer.

NOTE: The equipment must be used strictly in accordance with the Manufacturer's Guidelines.

Any alteration or repair carried out without the authority of iM3 will **VOID ALL WARRANTIES.**

9. TECHNICAL PRODUCT DETAILS

iM3 the company meeting the Veterinary Professionals Dental needs. The only company that designs, manufactures, sells and services its own equipment, that's how we maintain our reputation for quality, reliability and customer satisfaction, and we're proud of that so we work hard to maintain that reputation.

MANUFACTURED & SERVICED BY:

iM3 Dental Limited | EU

Unit 29, Duleek Business Park, Duleek, Co. Meath, A92 N72W, Ireland.

Phone: IRL: +353 (0) 1 691 1277 | UK: +44 (0) 142 322 4297

Email: sales@im3dental.com | support@im3dental.com

Website: www.im3vet.com

iM3 Inc. | US

12414 NE 95th St., Vancouver, WA, 98682, USA.

Phone: +1 800 664 6348 | Fax: +1 360 254 2940

Technical Support: +1 800 664 6348

Email: info@im3usa.com | Website: www.im3vet.com


iM3 Pty Ltd. | AU

21 Chaplin Drive, Lane Cove, NSW 2066, Australia.

Phone: +61 2 9420 5766 | Fax: +61 2 9420 5677

Email: sales@im3vet.com | support@im3vet.com

Website: www.im3vet.com.au

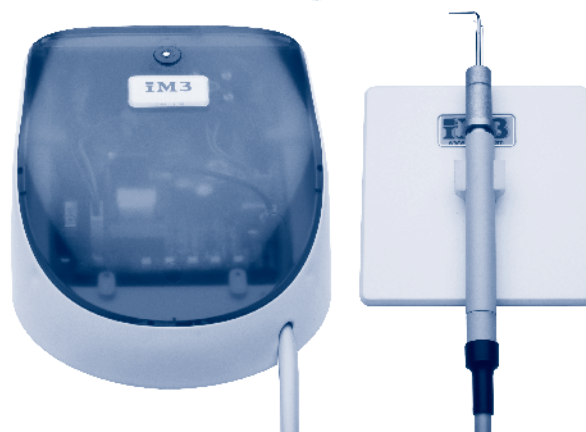SPECIFICATIONS

PRODUCT NAME	MAINS CONNECTION TYPE	MAINS VOLTAGE	MAINS FREQUENCY
iM3 42-12 Veterinary Dental Scaler	Single phase with protective grounding.	100 to 240 Vac (at option)	50/60 Hz
PRIMARY FUSES	CLASSIFICATION	MAX. POWER CONSUMPTION	WATER SUPPLY PRESSURE
400 mA 250 V slow blow	Class 1, type B (IEC 601-1)	20W (20VA)	1 to 3 bar (15-42 PSI)

10. LEGAL INFORMATION

MANUFACTURED BY:

iM3 Pty Ltd.,

21 Chaplin Drive, Lane Cove, NSW 2066, AUSTRALIA.

Phone: +61 2 9420 5766 | Fax: +61 2 9420 5677

Email: sales@im3vet.com | support@im3vet.com

Website: www.im3vet.com.au

MANUFACTURERS LIABILITY

iM3 is liable for material or manufacturing defects within a warranty period of Twelve months from the date of purchase, for replacement of parts that are found to be defective. This does not include broken stacks (inserts), or broken handpiece tips.

UNAUTHORIZED INTERFERENCE

iM3 cannot accept responsibility for damage caused by incorrect handling or repairs carried out by the third parties not authorized to do so by iM3!

CLAIMS UNDER WARRANTY

Claims under warranty accompanied by proof of purchase – must be sent to the vendor or to an authorized iM3 service organization. The provision of service under warranty extends neither the warranty period nor any other guarantee period.

TRANSPORT FOR REPAIR

Freight to and from the Repair Centre will be the Owners responsibility.

COPYRIGHT INFORMATION

The iM3 42-12 has been specifically created and designed by Philip Bloom of iM3 Australia, exclusively for Veterinary Dental use. No other ultrasonic dental scaler meets the specific needs of the Veterinary Dental Professional as does the 42-12.

Graphics, industrial design and Engineering Drawings were produced by iM3 Australia.

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