

iM3 Self Cure

Two-part Auto Mixing Cartridge
For veterinary use only



iM3 Self Cure is a temporary composite material for temporary crowns and jaw fracture repairs etc.

Characteristics

- Strong material that doesn't break even when thin.
- No smell.
- Rapid polymerization means short working time.
- Flows smoothly thanks to mixing tip.
- 50ml 1:1 cartridge for use with iM3 Applicator Gun - Order code APP101
- Possible to use with light body mixing tip.

Application Range

Temporary single crown and jaw fracture repairs etc.

Application method

- For use with the iM3 application gun
- Attach the supplied mixing tip
- Complete polymerization takes approx. 6 ~ 7min

Cautions

- iM3 Self Cure is only used for veterinary dental applications.
- Avoid contact with eyes.
- In case of contact with the skin or eyes, clean with pure water several times, and then consult with a medical specialist if necessary.
- Do not use expired iM3 Self Cure.
- For Veterinary use only.

Storage

- Do not store under intense light.
- Close the cap immediately after use.
- Store in a dry and cool place.
- Valid for 2 years from the manufacturing date.

Manufactured for

iM3[®]

THE VETERINARY DENTAL COMPANY

USA (800) 664 6348 info@im3usa.com

Australia +61 2 9420 5766 sales@im3vet.com

Ireland +353 (0)1 6911277 sales@im3dental.com